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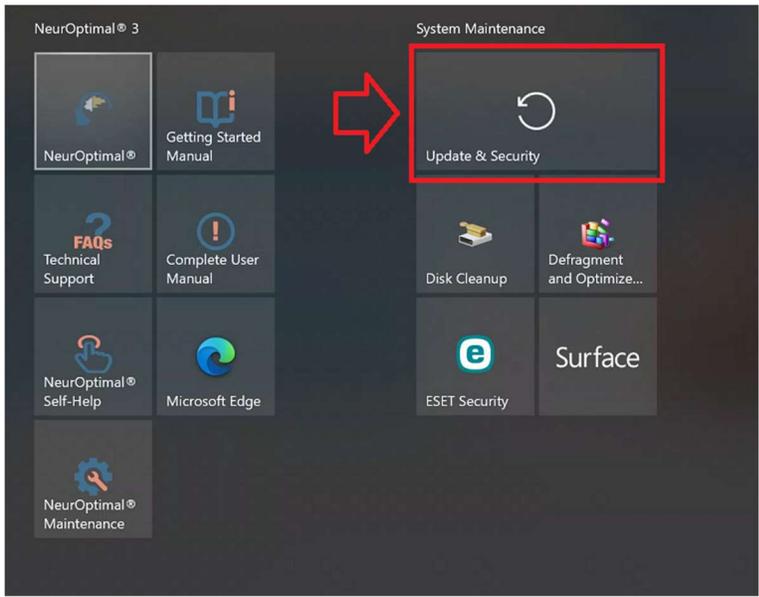
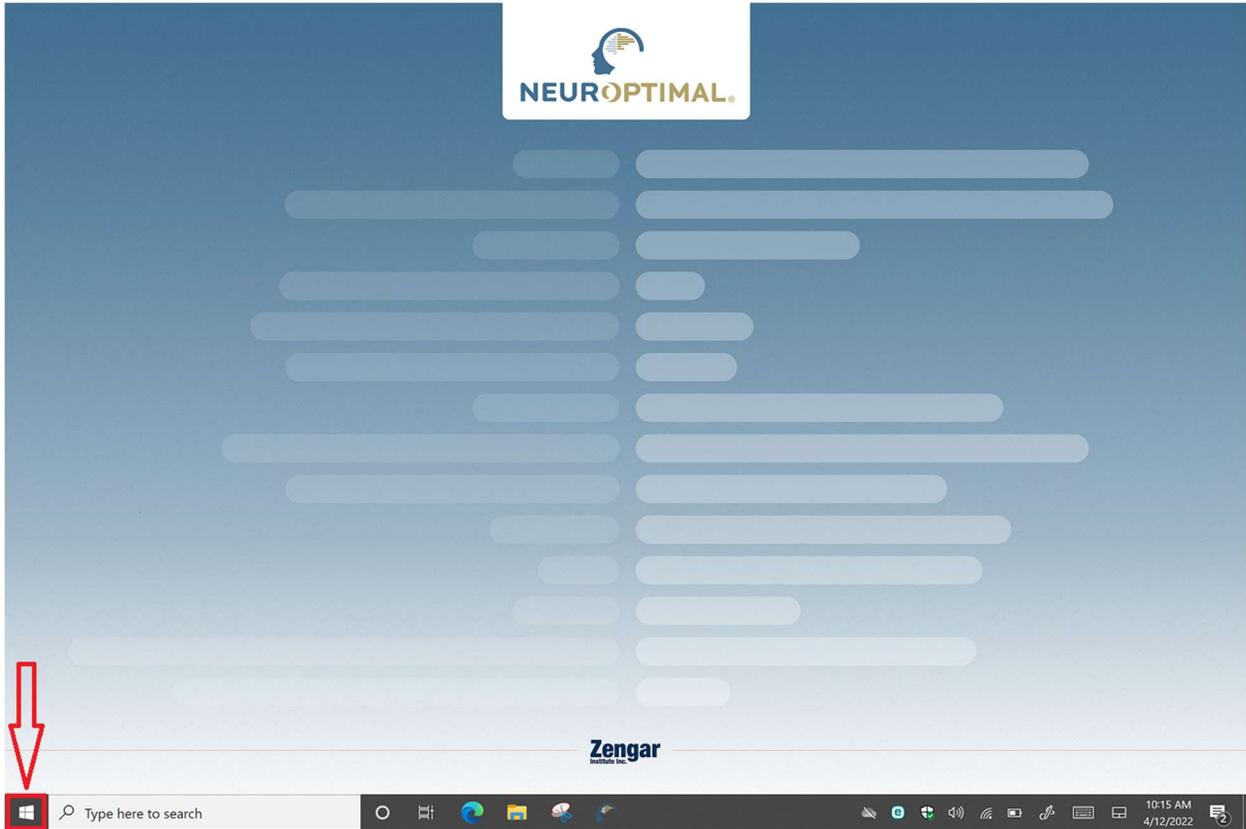
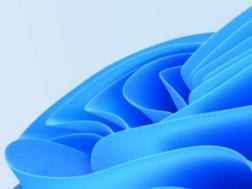
Windows Update

Upgrade to Windows 11 is ready—and it's free!

Get the latest version of Windows, with [a new look, new features, and enhanced security.](#)

[Download and install](#) Stay on Windows 10 for now

Looking for info on the latest updates?

[Learn more](#)

Related links

[Check Storage](#)

[OS build info](#)

[Get help](#)

[Give feedback](#)

You're up to date
Last checked: Today, 2:05 PM

[Check for updates](#)

 **Pause updates for 7 days**
Visit [Advanced options](#) to change the pause period

 **Change active hours**
Currently 8:00 AM to 5:00 PM

 **View update history**
See updates installed on your device

 **Advanced options**
Additional update controls and settings

Windows Update

Upgrade to Windows 11 is ready—and it's free!

Get the
a new

Down

Check fo

Pat

Vis

Ch

Cu

View

See

Ad

Ad

Software License Terms

Last updated June 2021

MICROSOFT SOFTWARE LICENSE TERMS

WINDOWS OPERATING SYSTEM

IF YOU LIVE IN (OR IF YOUR PRINCIPAL PLACE OF BUSINESS IS IN) THE UNITED STATES, PLEASE READ THE BINDING ARBITRATION CLAUSE AND CLASS ACTION WAIVER IN SECTION 11. IT AFFECTS HOW DISPUTES ARE RESOLVED.

Thank you for choosing Microsoft!

Depending on how you obtained the Windows software, this is a license agreement between (i) you and the device manufacturer or software installer that distributes the software with your device; or (ii) you and Microsoft Corporation (or, based on where you live or, if a business, where your principal place of business is located, one of its affiliates) if you acquired the software from a retailer. Microsoft is the device manufacturer for devices produced by Microsoft or one of its affiliates, and Microsoft is the retailer if you acquired the software directly from Microsoft. If you are a volume license customer, use of this software is subject to your volume license agreement rather than this agreement.

This agreement describes your rights, obligations, and the conditions upon which you may use the Windows software. You should review the entire

To install this update, accept the license terms. Close this window if you don't want to install the update now.

Accept and install

Close

Windows Update

Updates available

Last checked: Today, 2:05 PM

Upgrade to Windows 11

Status: Downloading - 0%

Pause updates for 7 days

Visit Advanced options to change the pause period

Change active hours

Currently 8:00 AM to 5:00 PM

View update history

See updates installed on your device

Advanced options

Additional update controls and settings

Looking for info on the latest updates?

[Learn more](#)

Related links

[Check Storage](#)

[OS build info](#)

[Get help](#)

[Give feedback](#)

Windows Update ×

Windows 11 is downloading
We'll let you know when it's ready to install.

Windows Update

Restart required

This update is ready to install! We need your help deciding when to restart so we can finish up.

Upgrade to Windows 11

Status: Pending restart

Restart now

Schedule the restart

Pause updates for 7 days

Visit Advanced options to change the pause period

Change active hours

Currently 8:00 AM to 5:00 PM

View update history

See updates installed on your device

Advanced options

Additional update controls and settings

Looking for info on the latest updates?

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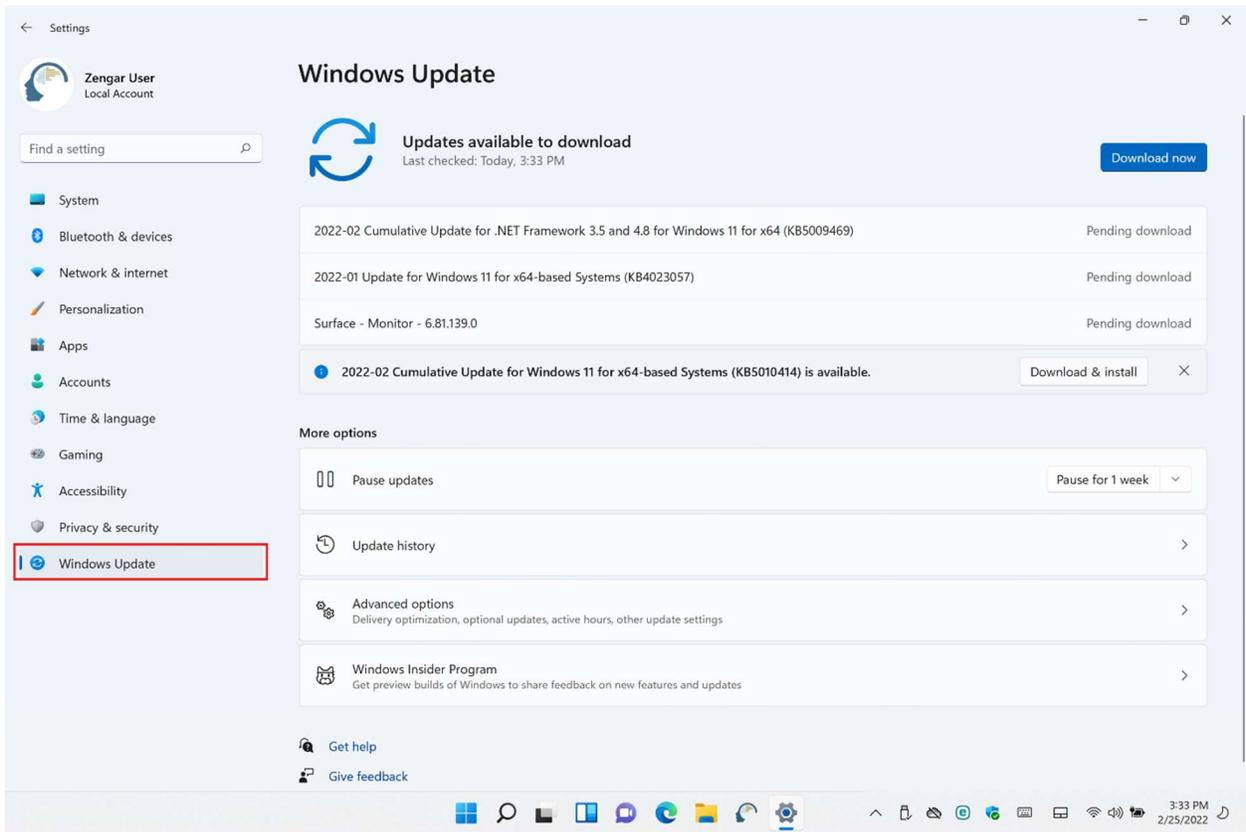
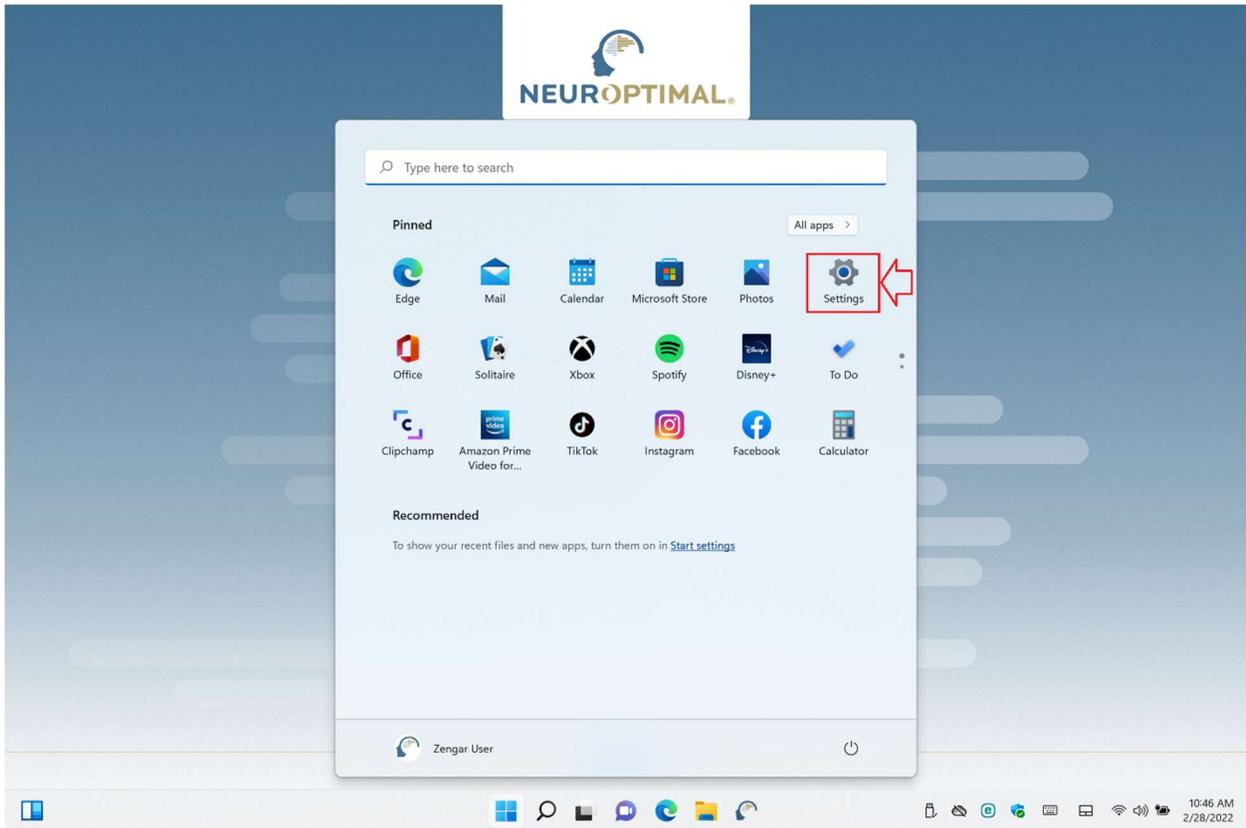
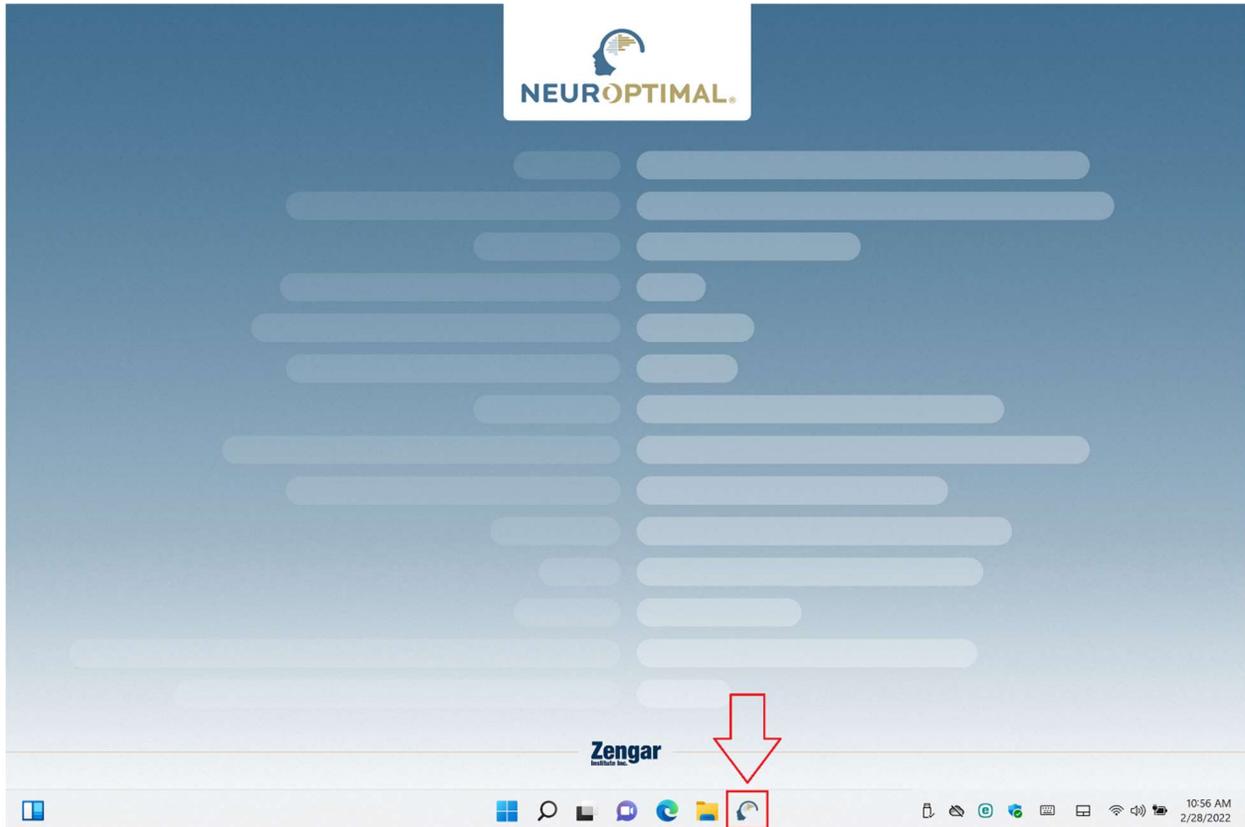
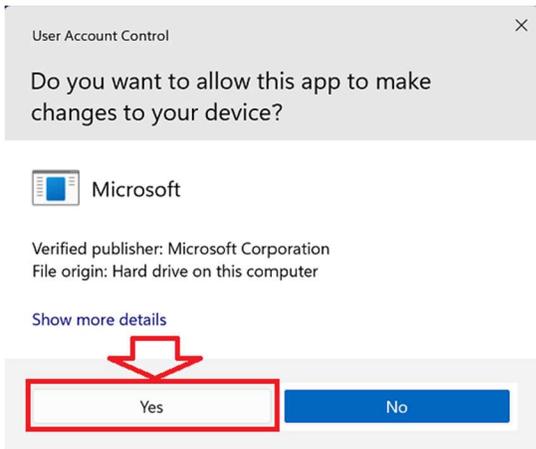
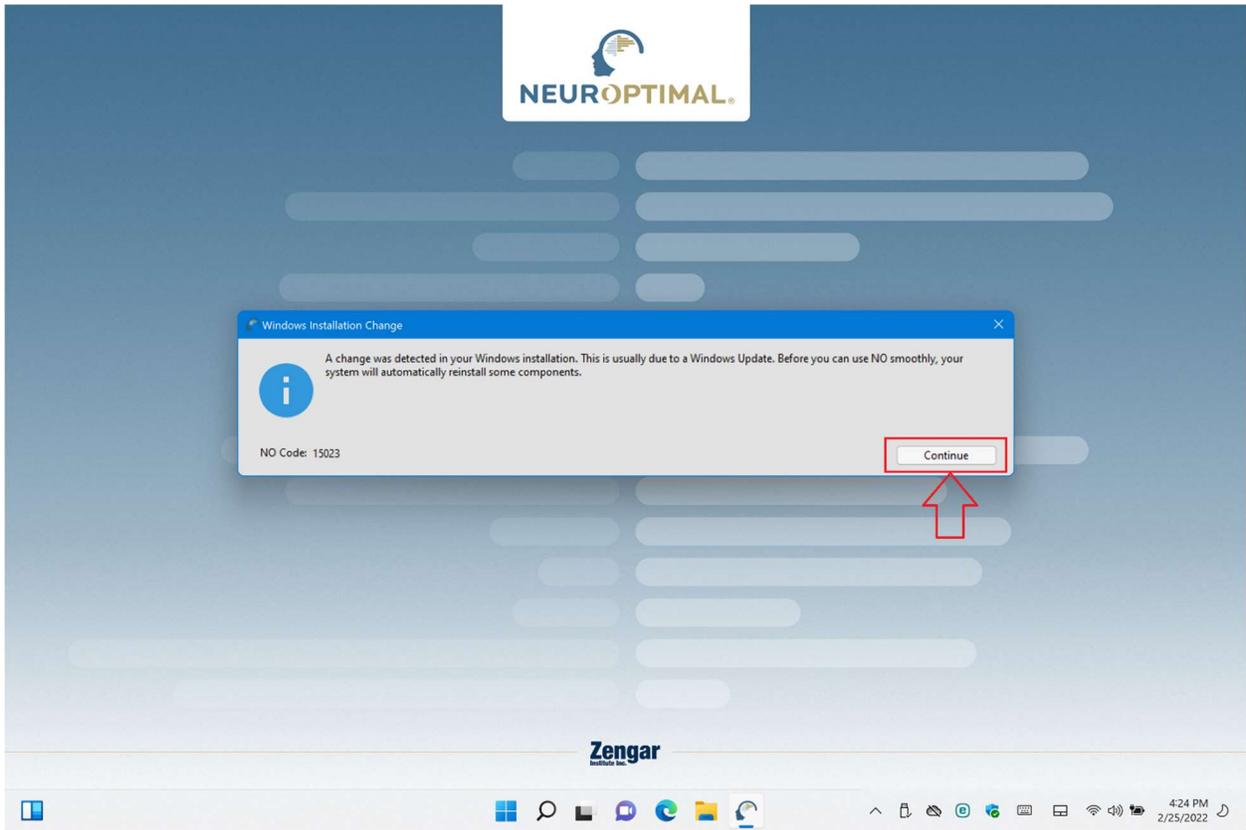
Maintain NeuroOptimal

Completing the NeuroOptimal Wizard

Click Finish to exit the NeuroOptimal Wizard.

Operation successful

It is strongly suggested that you reboot your computer before using NeuroOptimal.

Do you want to reboot your computer now?

After you click "Yes" please wait. It will take up to one minute before the computer restarts.

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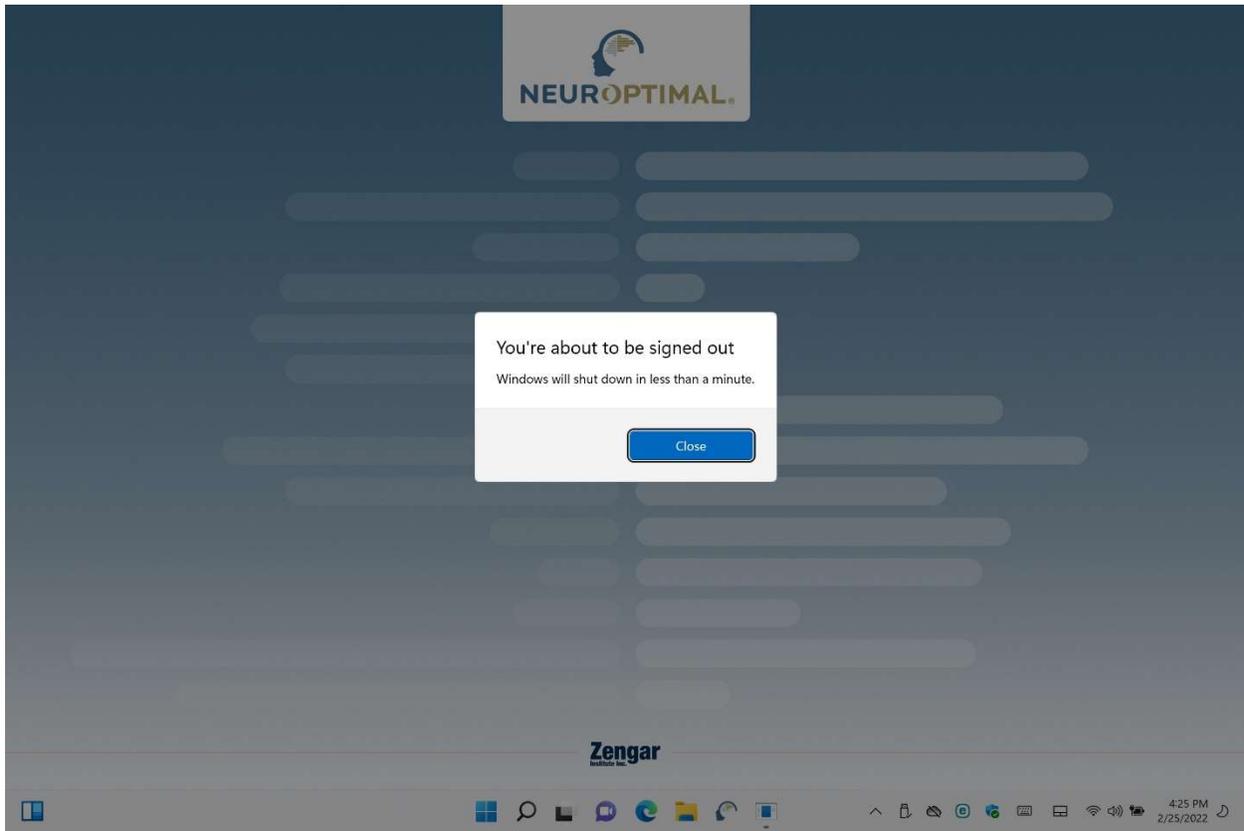
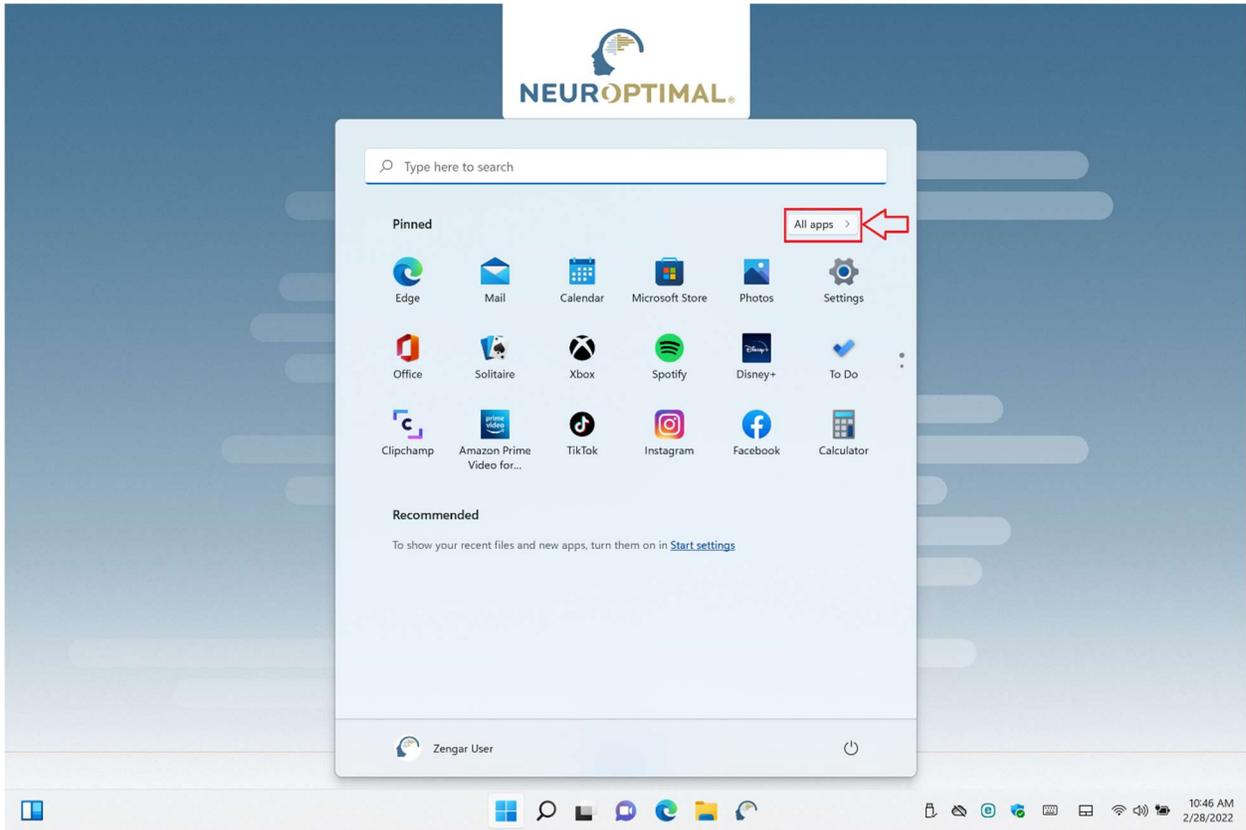
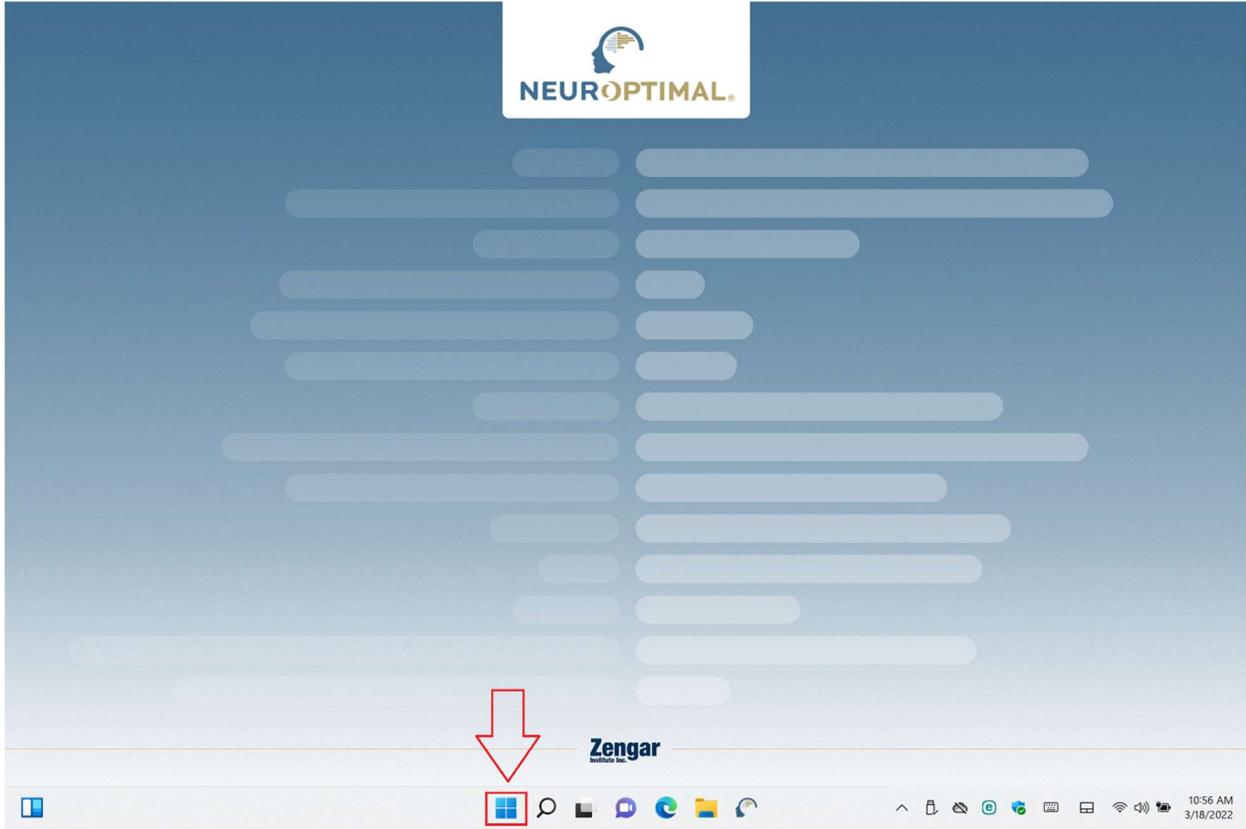
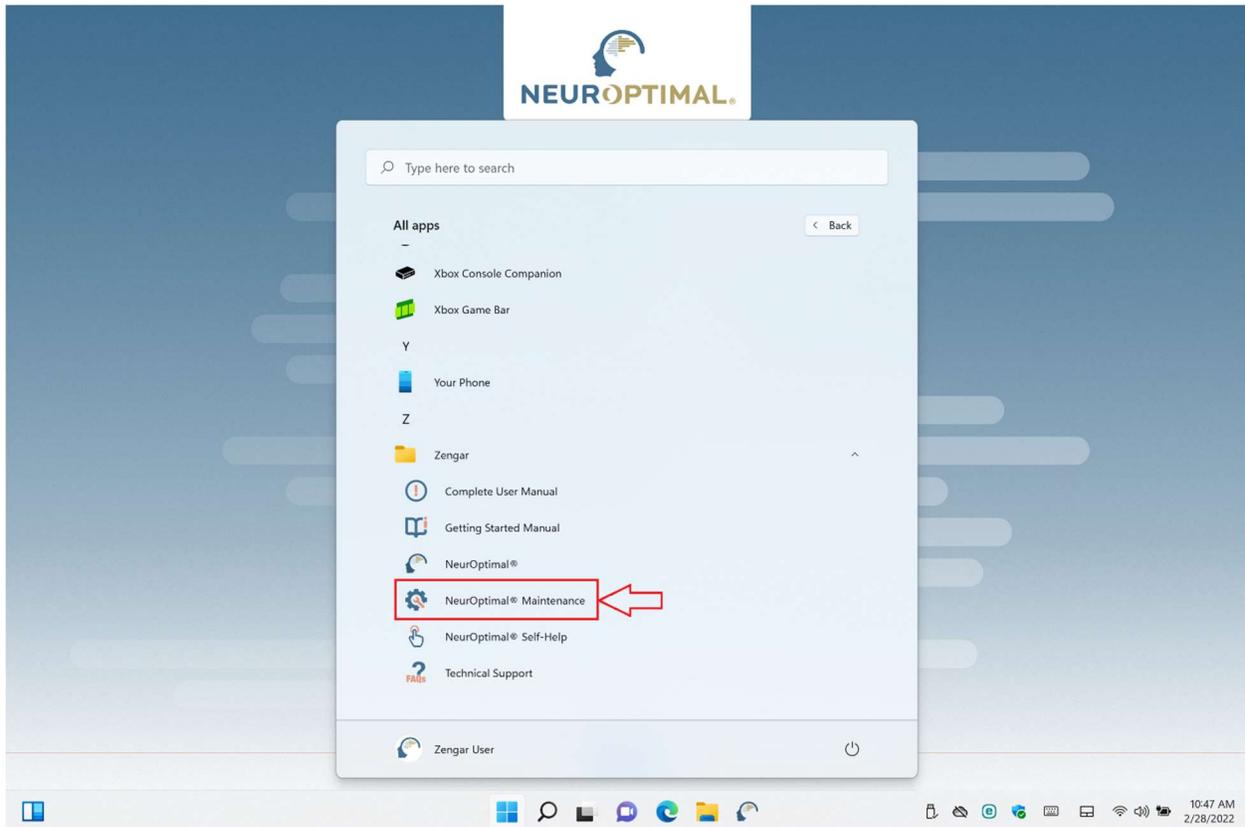
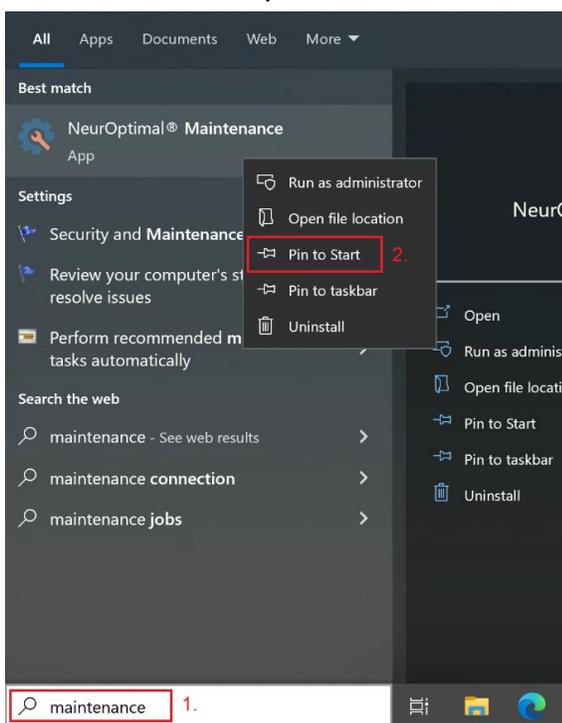
Maintain NeuroOptimal

Setup - NeuroOptimal

Welcome to the NeuroOptimal Setup Wizard.

1. Add or remove components

Update components

Remove all components

Settings 2.

Maintain NeuroOptimal

Select Components
Select the components to install. Deselect installed components to uninstall them. Any components already installed will not be updated.

Reset | Select All | Deselect All

Component Name	Act	
<input checked="" type="checkbox"/> Maintenance Tool	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Backup NO	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Beyond Trust - Remote Support Automatic Elevation Service	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> DigitAI Sessions	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> G-Force	<input checked="" type="checkbox"/>	Installs Zengar Driver on your computer.
> <input checked="" type="checkbox"/> Kodi	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> LabVIEW Run-Time Engine 2019 64-bit	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> License Agreements	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Maintenance Tool Icon	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Maintenance Tool Shortcut	<input checked="" type="checkbox"/>	
> <input checked="" type="checkbox"/> Manuals	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Media files	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Microsoft Access Database Engine 64-bit	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Migration Utility	<input checked="" type="checkbox"/>	
> <input checked="" type="checkbox"/> MySQL Components	<input checked="" type="checkbox"/>	This component will occupy approximately 2.75 MB on your hard disk drive.

< Back | Next > | Cancel

Maintain NeuroOptimal

Select Components
Select the components to install. Deselect installed components to uninstall them. Any components already installed will not be updated.

Reset Select All Deselect All

Component Name	Act	
<input checked="" type="checkbox"/> Maintenance Tool	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Backup NO	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Beyond Trust - Remote Support Automatic Elevation Service	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> DigitAI Sessions	<input checked="" type="checkbox"/>	
1. <input type="checkbox"/> G-Force	<input type="checkbox"/>	Installs Zengar Driver on your computer.
> <input checked="" type="checkbox"/> Kodi	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> LabVIEW Run-Time Engine 2019 64-bit	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> License Agreements	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Maintenance Tool Icon	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Maintenance Tool Shortcut	<input checked="" type="checkbox"/>	
> <input checked="" type="checkbox"/> Manuals	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Media files	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Microsoft Access Database Engine 64-bit	<input checked="" type="checkbox"/>	
<input checked="" type="checkbox"/> Migration Utility	<input checked="" type="checkbox"/>	
> <input checked="" type="checkbox"/> MySQL Components	<input checked="" type="checkbox"/>	This component will occupy approximately 2.75 MB on your hard disk drive.

< Back 2. **Next >** Cancel

Maintain NeurOptimal ×

Ready to Update Packages

Setup is now ready to begin updating your installation. Installation will use 0.00 bytes of disk space.

< Back **Update** Cancel

User Account Control ×

Do you want to allow this app to make changes to your device?

 Microsoft

Verified publisher: Microsoft Corporation
File origin: Hard drive on this computer

[Show more details](#)

Yes No

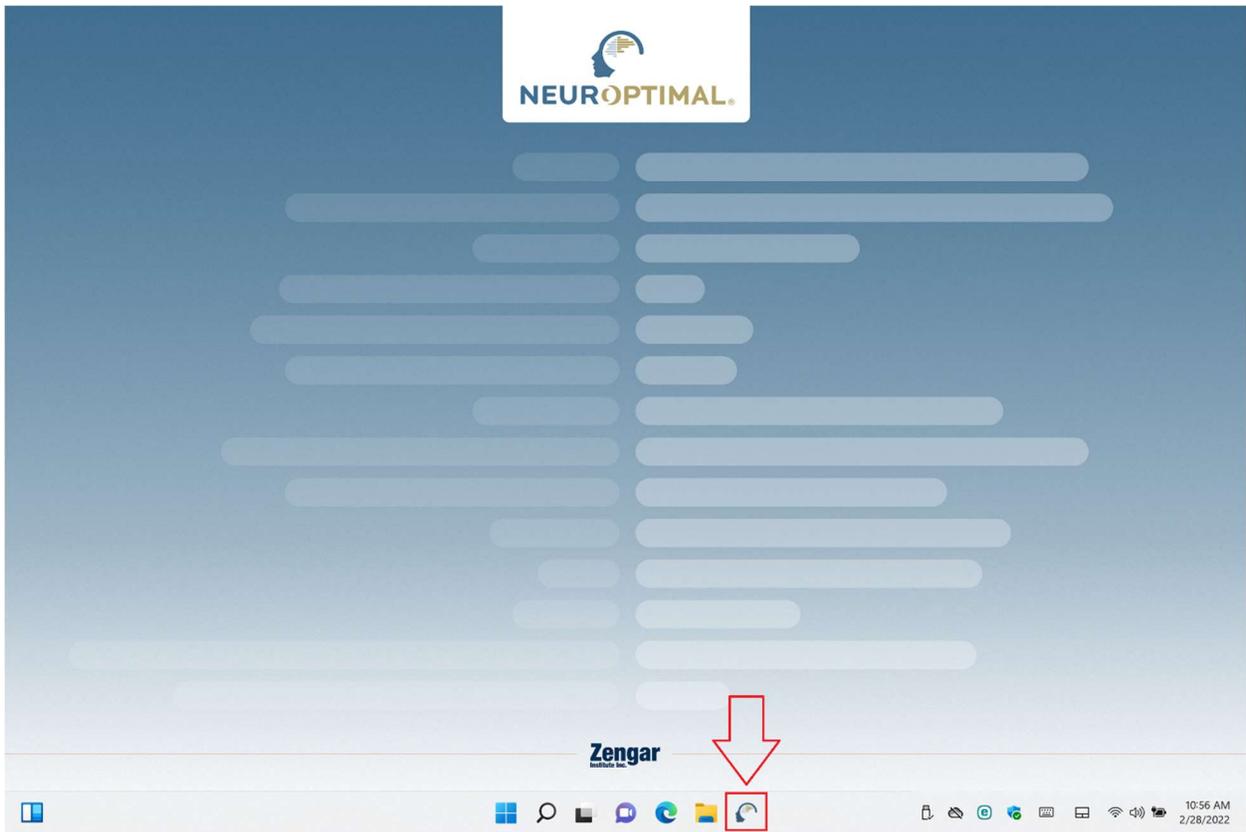
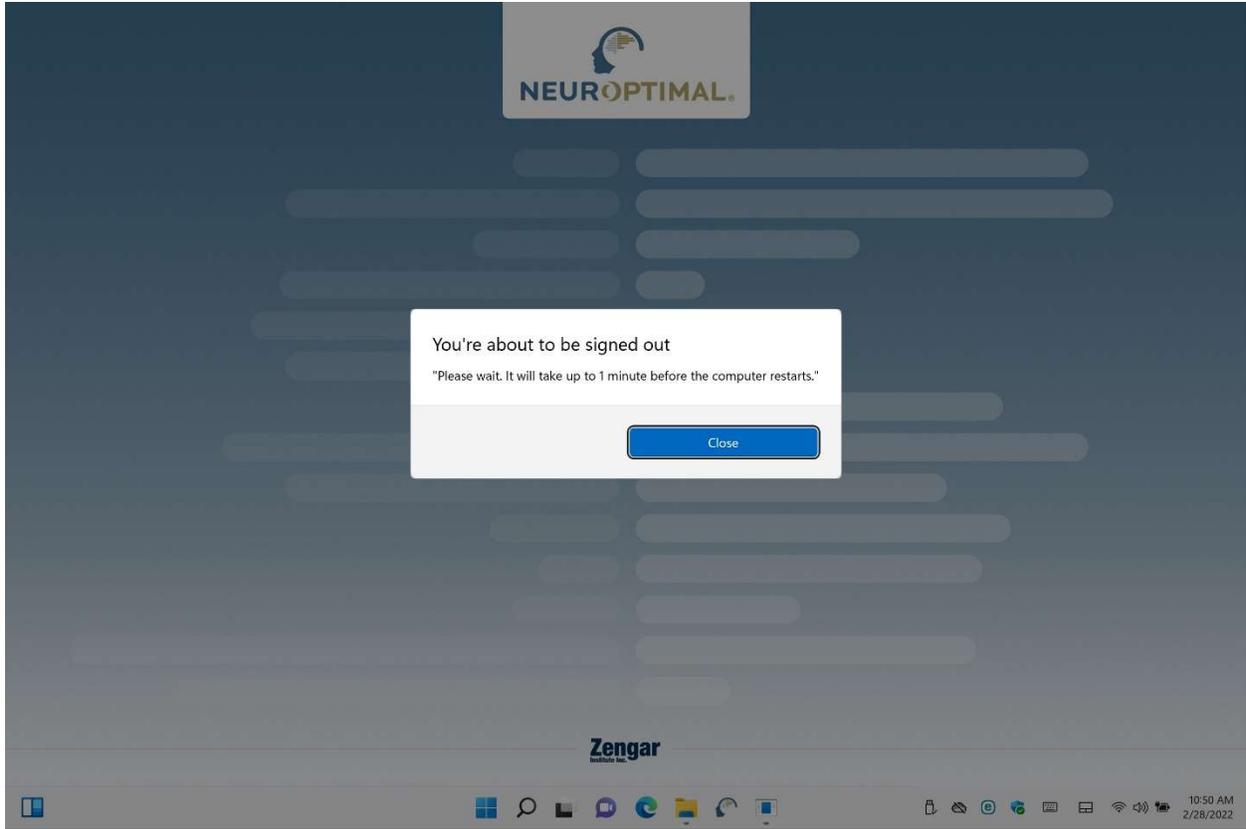
Maintain NeurOptimal

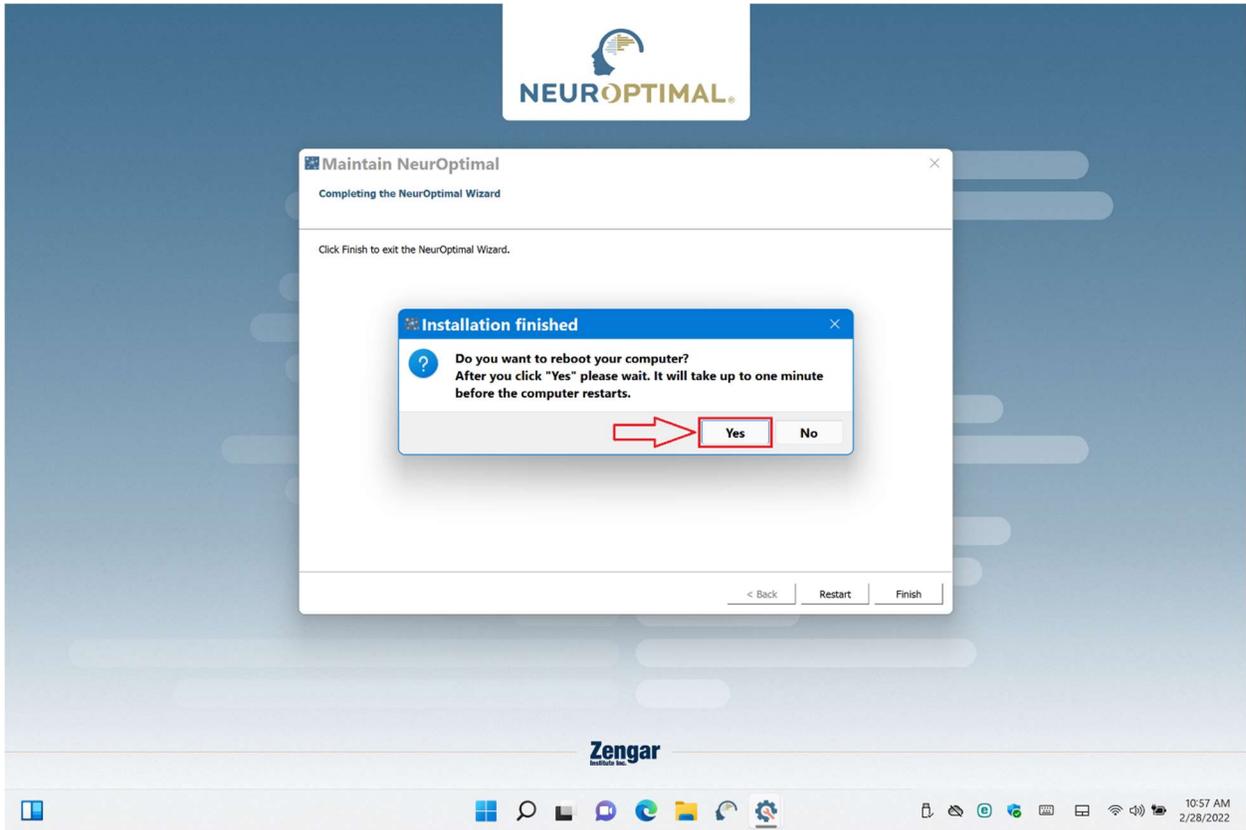
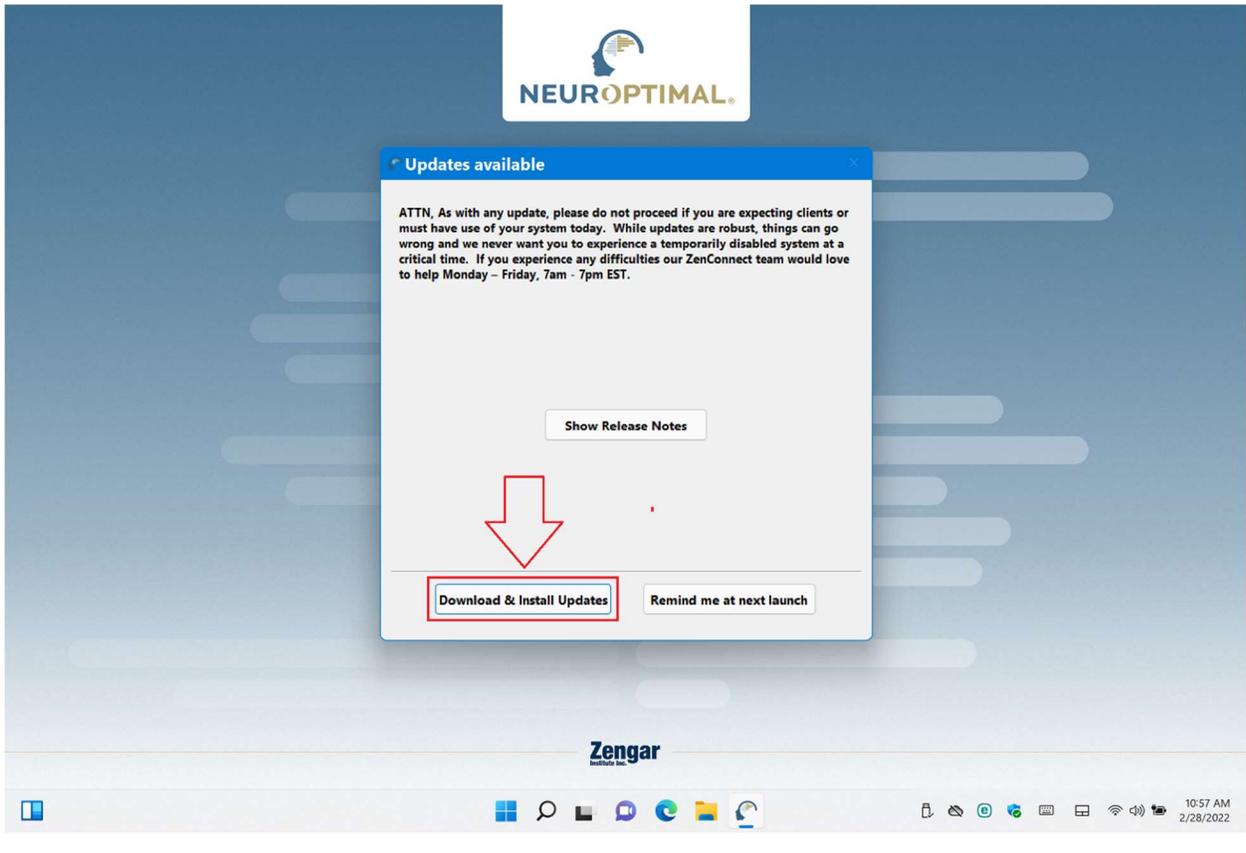
Completing the NeurOptimal Wizard

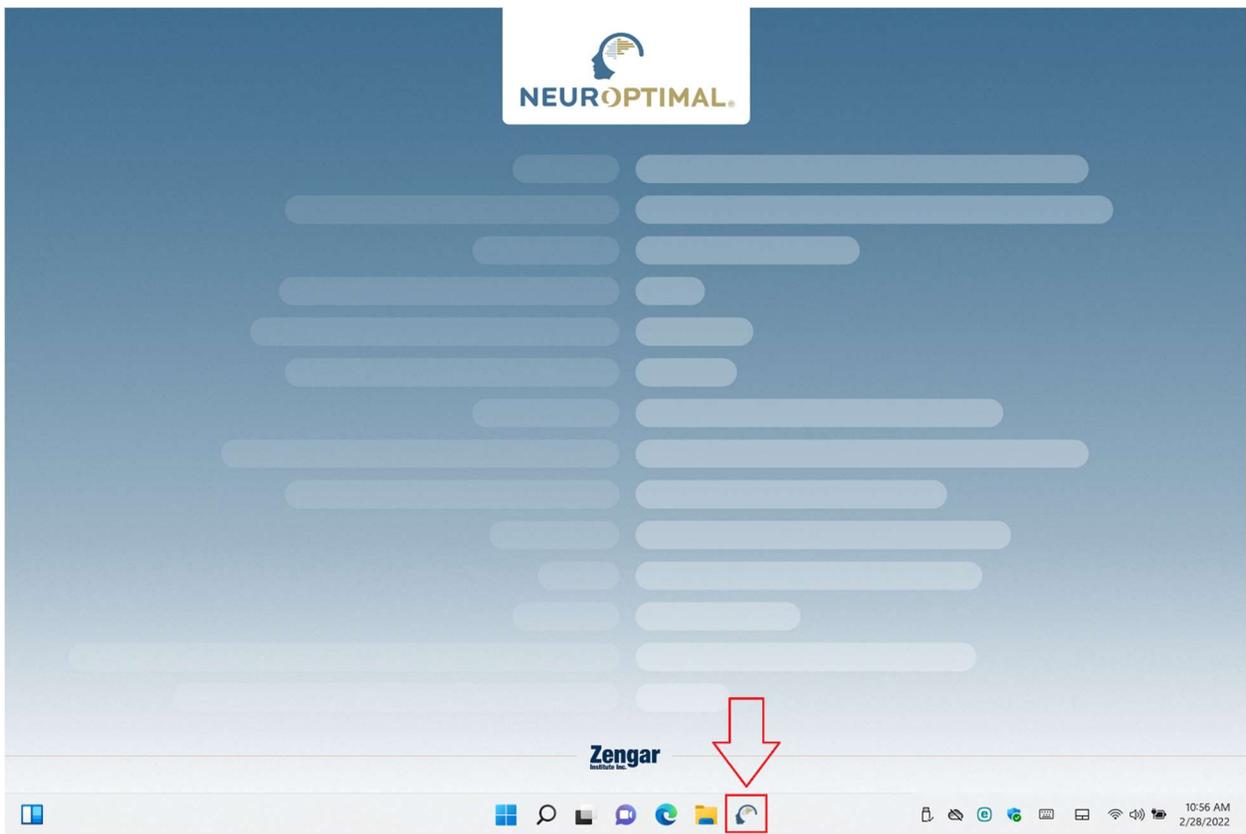
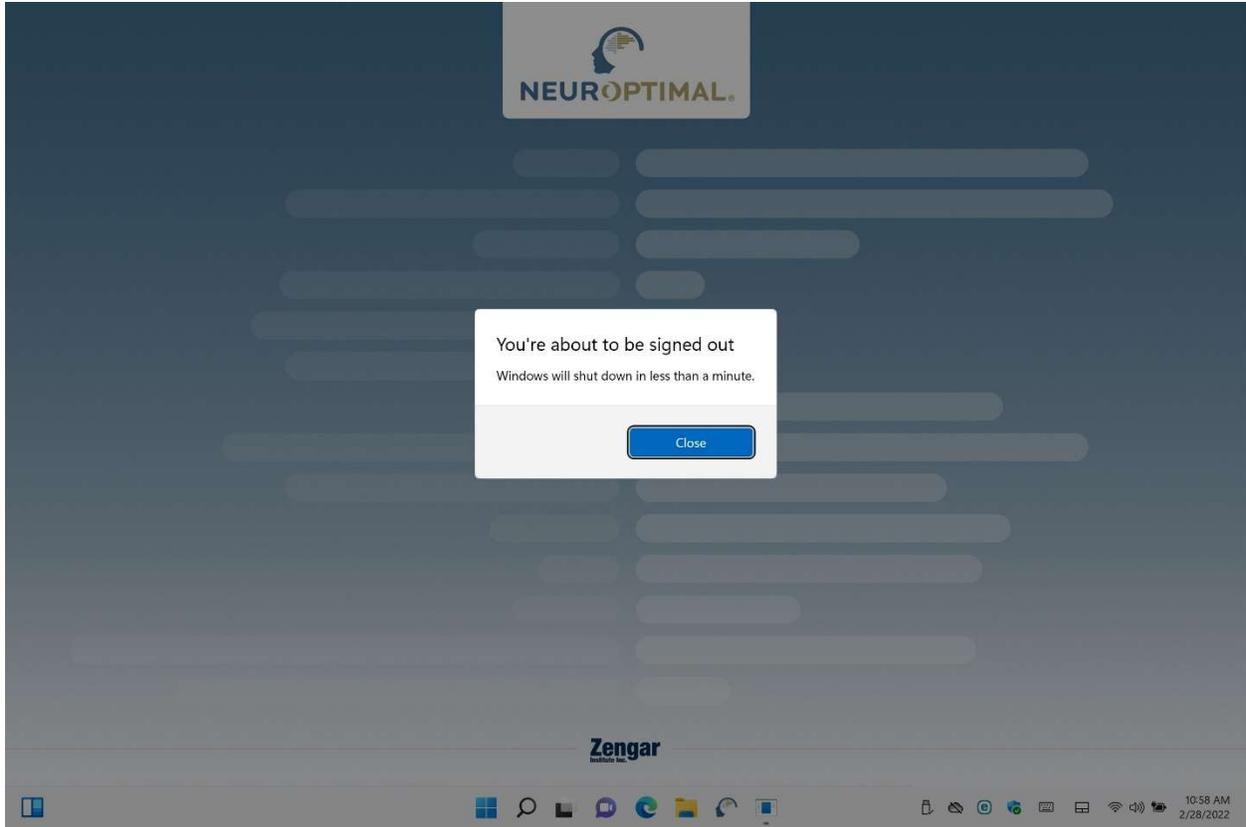
Click Finish to exit the NeurOptimal Wizard.

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